



# PROBENVORBEREITUNG UND PRÜFVERFAHREN

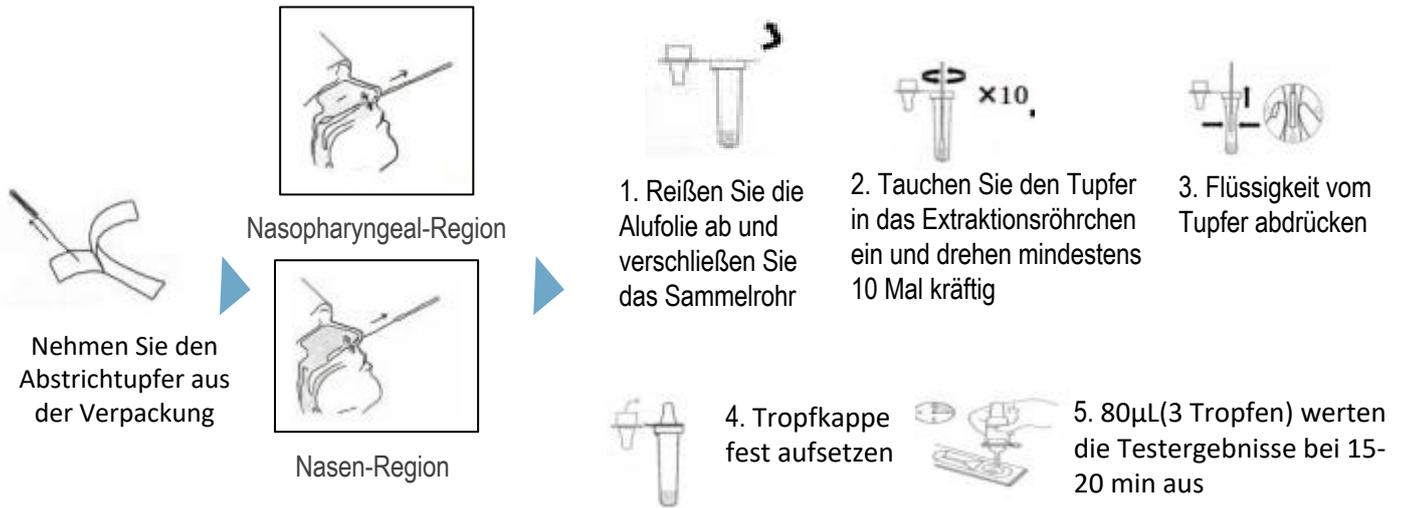

# INTERPRETATION DER TESTERGEBNISSE

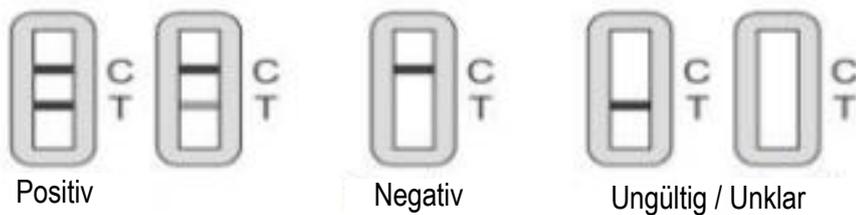

# KLINISCHE MANIFESTATIONEN

Nasopharyngealer Abstrich				
	CT≤33		CT≤37	
Sensibilität	97.06% (94.29%-98.72%)	264/272	93.68% (90.20%-96.21%)	267/285
Spezifität	99.68% (98.24%-99.99%)	314/315	99.68% (98.24%-99.99%)	314/315
Nasenabstrich				
	CT≤33		CT≤37	
Sensibilität	96.32% (93.34%-98.22%)	262/272	92.98% (89.37%-95.66%)	265/285
Spezifität	99.37% (97.73%-99.92%)	313/315	99.37% (97.73%-99.92%)	313/315



Do not freeze





**Suzhou Soochow University Saier Immuno Biotech Co.,Ltd**

Add:No.502 Zhongnan road ,Suzhou Industrial Park,

Jiangsu, China

Tel:86-512-62831303

Web:www.saiertech.com

## Letter of Declaration

We, Suzhou Soochow University Saier Immuno Biotech Co.,Ltd.(hereinafter"SAIERTECH"),with the address of No.502 Zhongnan road ,Suzhou Industrial Park, Jiangsu, China,hereby declare that our product "InstantSure Covid-19 Ag CARD " and its variants are compatible with the new virus strain B.1.1.7,501Y.V2, 501Y.V3, B.1.617.2 and B.1.529.

Saiertech commits to continue the evaluation of test kit's compatible against new SARS-COV-2 variants and will keep our distributors and customers updated.

If you have any questions or concerns, please feel free to contact us.

Sincerely Yours.

Suzhou Soochow University Saier Immuno Biotech Co.,Ltd.



# BfArM / Paul-Erich-Institut Covid-19 Antigentest Liste

Pro Seite 10

Profi / Selbsttest AI

Cq ≤25 Alle

Cq 25-30 Alle

Cq ≥30 Alle

Gesamt- Sensitivität

Suchen Sie nach Testname, AT-Nr., Ref-Nr., Hersteller oder Land

Mit geprüft (0)

Sie wählen derzeit 0 Antigentests aus. Alles 199 auswählen? [Alles auswählen](#)

<input type="checkbox"/>	#	AT-Nr. / AT-No.	Profi / Selbsttest	Hersteller / Manufacturer	Land / Country	Testname / Test name	Cq ≤25	Cq 25-30	Cq ≥30	Gesamt- Sensitivität / total sensitivity	Details
<input type="checkbox"/>	1	AT1054/21	Profitest	Lifecosm Biotech Limited	CN	COVID-19 Rapid Test Cassette Antigen Test Kit	100%	100%	100%	100%	
<input type="checkbox"/>	2	AT731/21	Profitest	Guangdong Longsee Biomedical Co. Inc.	CN	Longsee 2019-nCoV Ag Rapid Detection Kit (Immuno-Chromatography)	100%	100%	100%	100%	
<input type="checkbox"/>	3	AT889/21	Profitest	CITEST Diagnostics Inc.	CA	COVID-19 Antigen Rapid Test (Oral Fluid)	100%	100%	90%	98%	
<input type="checkbox"/>	4	AT830/21	Profitest	Suzhou Soochow University Saier Immuno Biotech Co. Ltd.	CN	InstantSure Covid-19 Ag CARD	100%	100%	80%	96%	
<input type="checkbox"/>	5	AT961/21	Profitest	Hangzhou Jucheng Medical Products Co. Ltd	CN	SARS-CoV-2 Ag Schnelltestkit	100%	100%	80%	96%	
<input type="checkbox"/>	6	AT1024/21	Profitest	Acro Diagnostics Co. Ltd	US	SARS-CoV-2 Antigen Rapid Detection Kit (LFA)	100%	100%	80%	96%	
<input type="checkbox"/>	7	AT637/21	Profitest	Innova Medical Group Inc.	US	INNOVA SARS-CoV-2 Selbsttest (Qualitativer-Antigen-Schnelltest)	100%	100%	80%	96%	